


260 Saxon/Sarum stairlift

for curved stairs

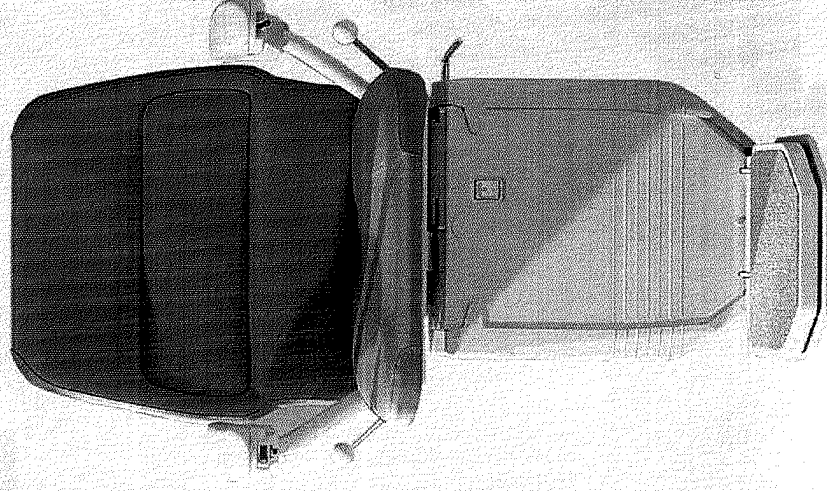

User guide
English



www.stannah.com

Stannah Stairlifts

Watt Close

East Portway

Andover

Hampshire

SP10 3SD

United Kingdom

Issue 1

April 06

Stannah suppliers

You can also find details at our website: www.stannah.com

UK Stannah branches
International suppliers

Birmingham	0121 559 2260	Australia	PR King & Sons Pty Ltd	Thailand	Thai Lifts
Brackley	01280 704 600		1300 859 267 (free call)		(66 2) 961 0422
Bristol	0117 955 9976		Tony Graystone	West Indies	RBP Lifts Ltd
Christchurch	01202 476 781		61 883 705 644		868 628 7058
Crayford	01322 555 777	Channel Islands	Rob Willder Lifts Ltd		
Gateshead	0191 460 0010		01534 739 515		
Glasgow	0141 882 9946	Iceland	Hedinn Schindler		
Mansfield	01623 631 010		5-444-777		
Norwich	01603 748 021	Malta	Lift Services Ltd		
Stockport	0161 477 3344		2125 2206		
		New Zealand	Independence Ltd		
			0800 92 66 77		
		Northern Ireland	Olympic Lifts Ltd		
			0845 330 5817		
		Republic of Ireland	Stannah Ireland		
			Daytime 01-450 7971		
			After hours 01-217 3786		

It is very unlikely that your stairlift will stop while travelling. However, if it does, we have provided you with a hand-winding tool and instructions on how to use it.

If your stairlift stops at the end of the rail and the green power light on the armrest goes off, use the checklist on page 19. If the checklist does not help you to solve the problem, please follow the instructions below.

If in any doubt, please contact your Stannah supplier for help. You will find their phone number on page 23 of this guide.


To move the stairlift using the hand-winding tool

Switch off the master power switch.

Remove the plastic cap on the front of the chair base.

Put the hand-winding tool into the hole where labelled.

Turn the handle as shown on the label on the chair base.

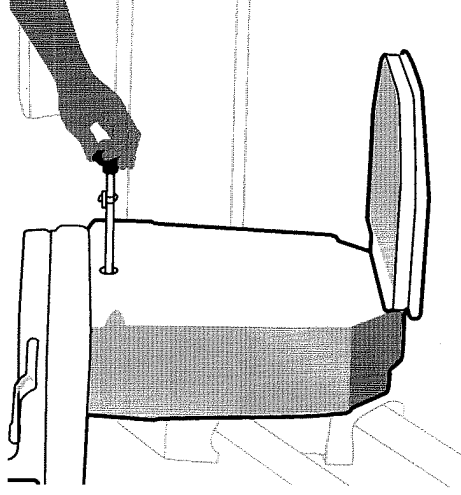

Safety

Never use the stairlift when the hand-winding tool is in the hole.

We have designed the 260 stairlift to help you continue enjoying your life at home.

To help you get the most from your new stairlift, please take a few minutes to read this guide. Then, please keep this guide in a safe place in case you need to check any details in the future.

If you feel unsure about any aspect of your stairlift, please do not hesitate to get in touch with your stairlift supplier. You will find their phone number on page 23 of this guide.

In this guide...

Getting to know your stairlift 4-7

Using your stairlift 8

Power key 8

Remote calling control 9

Unfolding the chair 10

Hinged rail 11

Seatbelt 12

Starting the stairlift 13

Swivelling the seat 14

Getting off the stairlift 15

Cleaning 17

Help 18


Changing the batteries 20

Technical 22



You will find safety information wherever you see the Safety! symbol.

Getting to know your stairlift

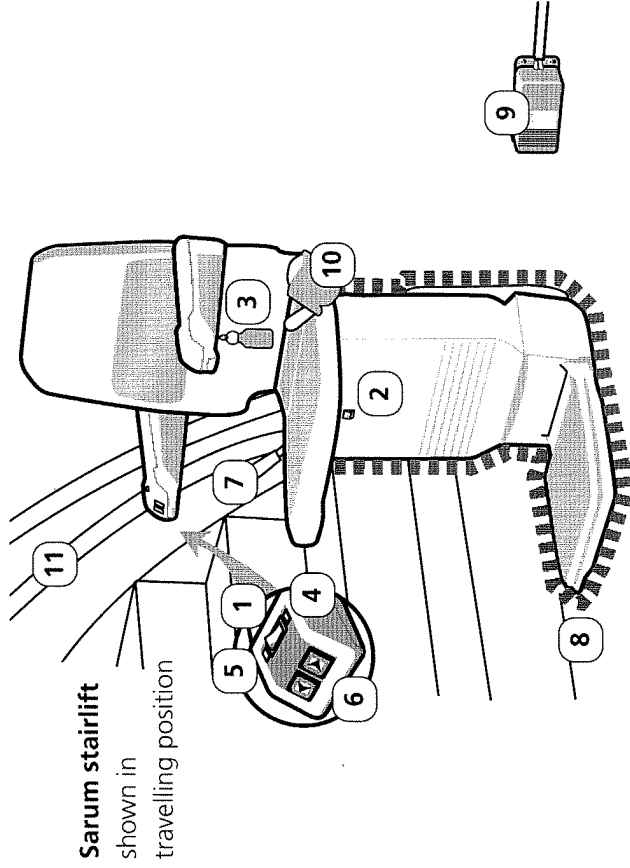

Sarum stairlift

shown in
travelling position

1 On/off switch

This should normally be left on.

See page 7 for more information.

2 Master power switch

5 Charge light

Yellow light is on when the stairlift is charging. (Saxon does not have a charge light.)

3 Power key

The keyhole is on the underside of the chair arm. The key must be inserted and turned to the on position before the stairlift will work.

6 Joystick or push button

Your stairlift may be fitted with a joystick or push button depending on the option you chose.

Moves the stairlift in the direction you want to travel.

If your chair has a powered swivel seat, you can also use the joystick or push button to swivel the seat.


4 Power light

The green power light comes on when the on/off switch and the power key are turned to the on position.

To change the battery

Slide down and open the battery cover, then insert the new batteries.

Each remote calling control takes three AA batteries.

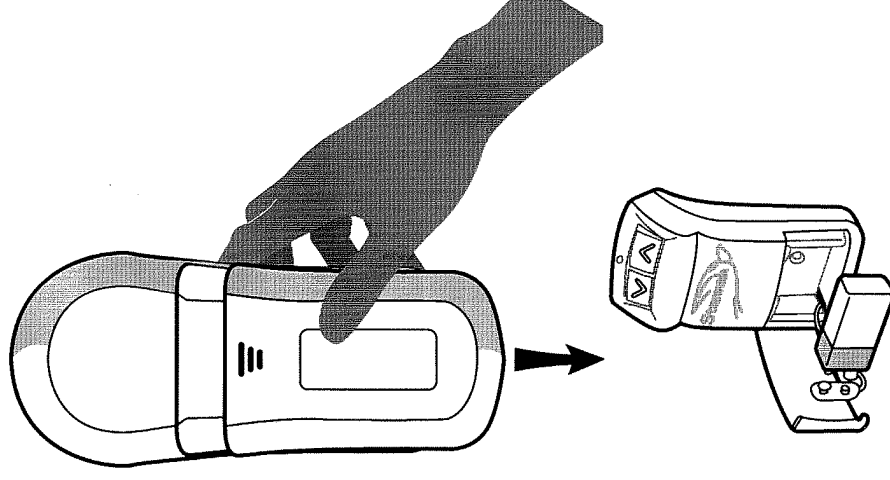

If you have a hinged rail, the battery compartment on your remote calling control is different.

Open the battery cover and insert a new 9V PP3 battery.

Do not use a Ni-cad rechargeable battery.

It is very unlikely that your stairlift will stop while travelling.

However, if it does, we have provided you with a hand-winding tool and instructions on how to use it.

If your stairlift stops at the end of the rail and the green power light on the armrest goes off, use the checklist on page 19. If the checklist does not help you to solve the problem, please follow the instructions below.

If in any doubt, please contact your Stannah supplier for help. You will find their phone number on page 23 of this guide.


To move the stairlift using the hand-winding tool

Switch off the master power switch.

Remove the plastic cap on the front of the chair base.

Put the hand-winding tool into the hole where labelled.

Turn the handle as shown on the label on the chair base.

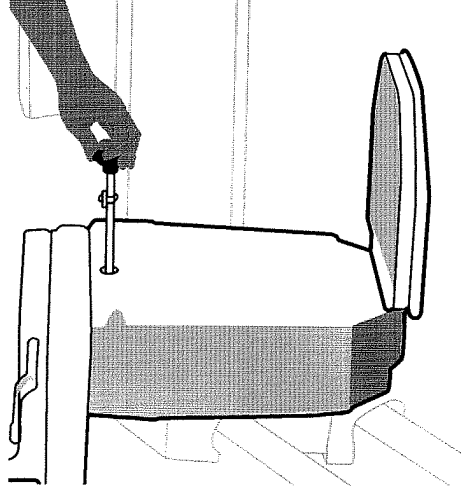

Safety

Never use the stairlift when the hand-winding tool is in the hole.

We have designed the 260 stairlift to help you continue enjoying your life at home.

To help you get the most from your new stairlift, please take a few minutes to read this guide. Then, please keep this guide in a safe place in case you need to check any details in the future.

If you feel unsure about any aspect of your stairlift, please do not hesitate to get in touch with your stairlift supplier. You will find their phone number on page 23 of this guide.

In this guide...

Getting to know your stairlift 4-7

Using your stairlift 8

Power key 8

Remote calling control 9

Unfolding the chair 10

Hinged rail 11

Seatbelt 12

Starting the stairlift 13

Swivelling the seat 14

Getting off the stairlift 15

Cleaning 17

Help 18

Changing the batteries 20

Technical 22



You will find safety information wherever you see the Safety! symbol.

Getting to know your stairlift



continued

Remote calling control

Positioned at the top and bottom of the staircase, the remote calling controls let you call the stairlift if it is at the opposite end of the staircase to you. The control sits in a holder that is attached to the wall, and can be used either when in its holder or when taken out.

The controls will not work unless the power key is turned to the on position.

The controls are battery powered. You will find information on how to change the battery on page 20.

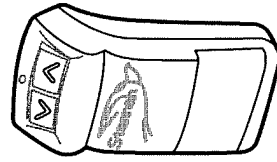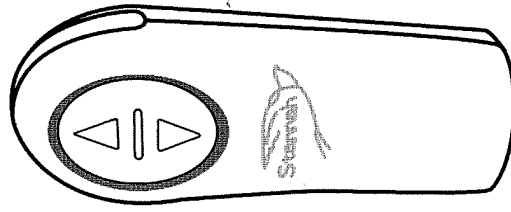

If you have a hinged rail, your remote calling control will look like this.

Stairlift will not move

Is the green light on?

No

- Check that the master power switch is switched on – see page 7.

The stairlift will not work if this switch is off.

- Check that the power key is inserted and turned to the on position – see page 8.

The stairlift will not work if the key is switched off.

Yes

- Check whether there is anything obstructing the safety edges – see picture.


If there is, remove the obstruction.

You may need to go in the opposite direction before you can move the obstruction and then continue to go up or down the stairs.

- Check that the chair is swivelled back to its travelling position. *The chair will not move unless it is in the travelling position – see pages 4 and 15.*

- Check the hinged rail is fully lowered.

The chair will not move if the hinged rail is raised – see page 11.

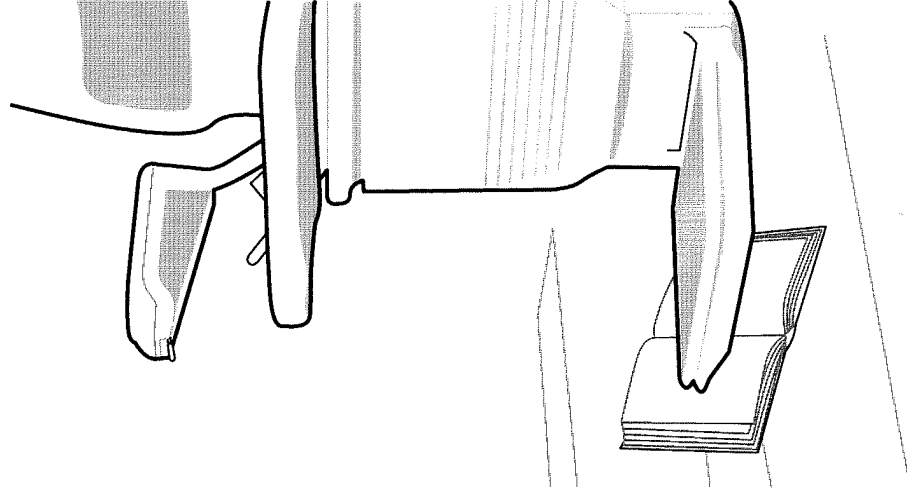

Changing the battery on the remote calling control


The remote calling control batteries will last about one year with average use.

When you press one of the arrow buttons, the light around the buttons will tell you how much power is in the battery.


Lights rotating	Batteries OK
Lights on constantly	Batteries low on power – change batteries as soon as possible
No light	Batteries have no power

If you have a hinged rail, the light on your remote calling control will work differently as follows:


Green light on	Batteries OK
Green light flashing	Batteries low on power – change batteries as soon as possible
No light	Batteries have no power

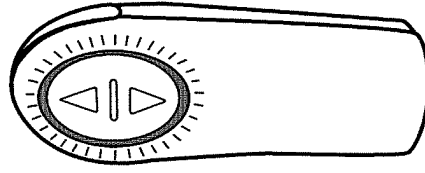

The lights are on constantly when the battery is low.

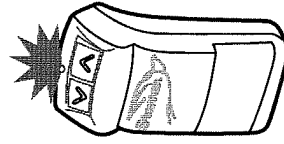

The green light is flashing when the battery is low.

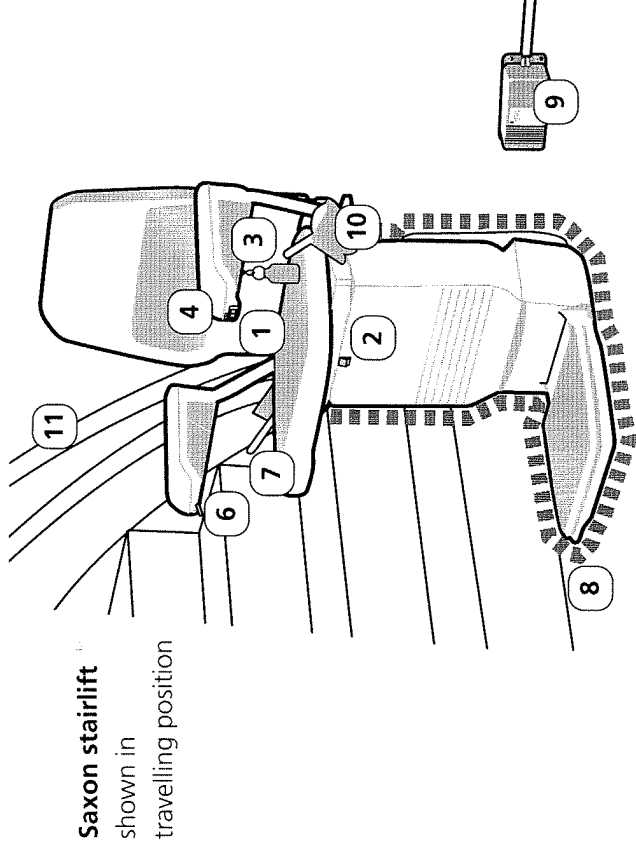

Saxon stairlift
shown in
travelling position

7 Swivel levers

These are on both sides of the seat. The seat will swivel outwards at the top of the stairs to help you get on or off the stairlift.

8 Safety edges

The safety edges are there to stop the stairlift if they come into contact with any obstacles. This stops anything from becoming trapped while the stairlift is being used.

9 Battery charger

Charges the batteries so the stairlift is always ready to travel. It also ensures the stairlift will work during a power cut.

This may be positioned at either the top or bottom of your staircase.

10 Seatbelt

There are various possible types of seatbelts. The one shown is a retractable lap seatbelt.

11 Rail

Using your stairlift


1 Turn on the power key

To turn on the power key


Put the key into the keyhole on the underside of the chair arm and turn it to the on position.

Switch the on/off switch to the on position.


The stairlift and the remote controls will not work unless you have turned on the power key and the on/off switch.

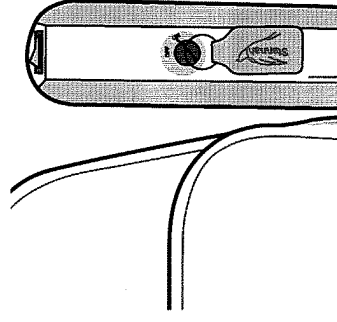

When the key and the on/off switch are in the on position, the green power light on the armrest will come on.

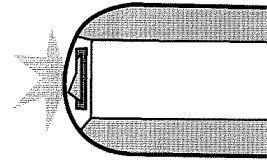

Saxon stairlift

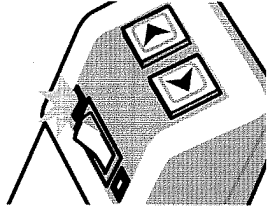


Sarum stairlift

To switch off the power key

Turn the key to the off position and take the key out.

The green power light will go off.

If you do not want to remove the key, leave the key in the off position as shown.

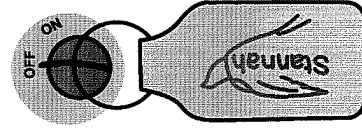

Cleaning your stairlift

If there are any aspects of cleaning your stairlift that you are unsure about, please call your Stannah supplier. You will find their phone number on page 23 of this guide.

Do not use any solvent cleaners, bleaches, abrasives, synthetic detergents, wax silicone polishes or aerosol sprays without checking with your stairlift supplier first.

To clean the rail

Use a damp cloth to remove any dust or dirt.

To clean the upholstery

Use a damp cloth to remove any dust or dirt.

In the unlikely event that you ever experience a problem with your stairlift, please use the following checklist.

If the checklist doesn't help you to solve the problem, please contact your stairlift supplier – their phone number is on page 23 of this guide.

Stairlift is beeping

The battery charger is not charging the batteries.

- Check that the mains power switch is on.

The position of the mains power switch can be found on page 2 of this guide.

- Check that the stairlift is at the top or bottom of the stairs.
- Check that there has not been a power cut.

If you cannot correct the fault, you can switch off the beeper by turning the master power switch to the off position. Please call your Stannah supplier.

Automatic swivel seat does not work

There may be a mechanical fault.


- Please call your stairlift supplier. *You will still be able to use the standard swivel until our technician can visit you.*

Master power switch

This switch is on the front of the chair and should normally be left switched on. **The stairlift will not work if this switch is off.**

You can switch the master power switch off – perhaps if you go away on holiday and your stairlift will not be used for more than a week.

Remember to switch the master power switch back on when you return.

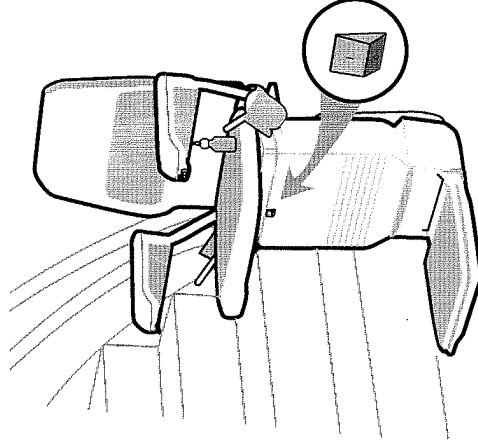

Options

You may have chosen from the range of options we provide for the Sofia and Solus stairlifts.

The options we've listed below are all covered in this guide, but if you need more information about any options that are not covered, please contact your local Stannah supplier. You'll find their phone number on page 23 of the guide.

Swivel seats

- standard swivel seat
- automatic swivel seat

Hinged rail

- standard hinged rail
- automatic hinged rail

Using your stairlift




2 Unfold the chair


Your chair can be folded away when it is not in use to give you more space around the top and bottom of your stairs.

To unfold the chair

To lower the footrest, use the lever on the side of the chair.

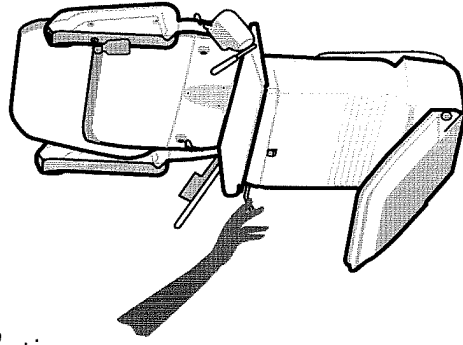

Lower both armrests.
Then lower the seat.

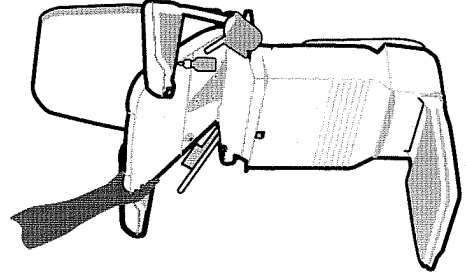

Safety

Stand with your feet clear of the footrest when you lower it



5 Get off the stairlift

Make sure the on/off switch is in the off position and the power light is off. Carefully, get off the stairlift and swivel the seat back to its travelling position. You can fold the chair away if you wish. *If you want to call the stairlift using the remote controls, make sure the on/off switch is in the on position.*


To get off

Unfasten the seatbelt.

Carefully get off the stairlift.




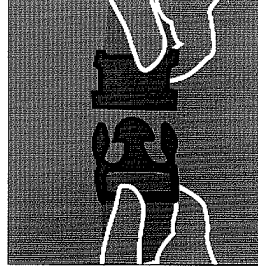
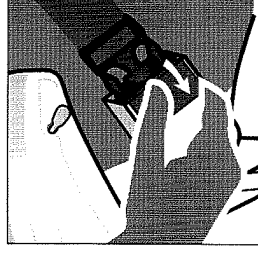


Swivel the seat back to its travelling position and fold the chair away if you wish.

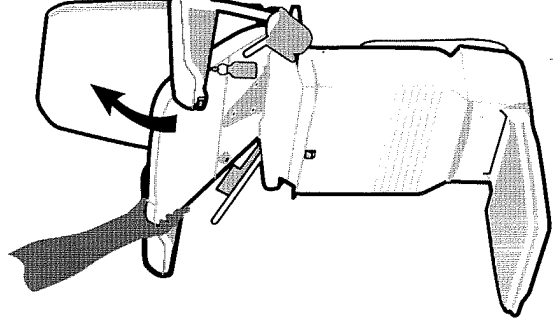

Safety

Follow these points to make sure you always use your stairlift safely.

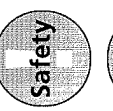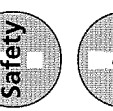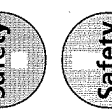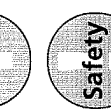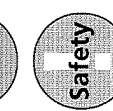-  **Safety**
It is unlikely that the stairlift will stop while you are using it. If it does, swivel the chair towards the top of the stairs using the standard swivel lever before you get off. If possible, swivel the seat back to its travelling position and fold the chair up before you make your way to the nearest end of the stairs.
-  **Safety**
Stand with your feet clear of the footrest when you lower it.
-  **Safety**
Always use the seatbelt.
-  **Safety**
Only undo the seatbelt when you have reached the top or bottom of the stairs and are ready to get off.
-  **Safety**
Keep your elbows tucked in close to your body when using the stairlift.
-  **Safety**
Only one person should ever use the stairlift at any time. Please see page 12 for maximum weight details.
-  **Safety**
Never use your stairlift in the event of fire.
-  **Safety**
Never use the stairlift when the hand-winding tool is in the hole – see page 22.
-  **Safety**
Never attempt to dismantle the stairlift.

Using the remote calling controls

Use these when you want to move the stairlift to the opposite end of the stairs.


To move the stairlift along the rail

Press and hold the arrow button on the remote calling control to move the stairlift in the direction you want. You can move the stairlift whether the chair is folded or unfolded.

There is information on how to unfold the chair on page 10.

If you release the arrow button, the stairlift will stop and will not start again for a few seconds.

This is a safety feature – wait for a few seconds before starting the stairlift again.

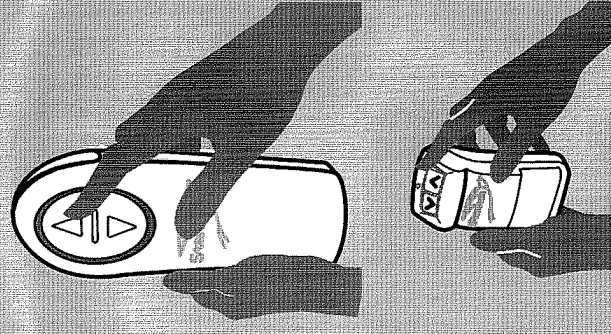

If you have a hinged rail, your remote calling control will look like this.

Battery status

When you press one of the arrow buttons, the light on the control shows how much power is in the battery.

You will find more information on how to check the battery on page 20.

The batteries will last about one year with average usage.

You will find information on how to change the batteries on page 20.

Using your stairlift




3 Settle into the seat and start the stairlift


Make sure the on/off switch is in the off position and the power light is off. Then, settle yourself into the seat, and put your seatbelt on. When you are ready to travel up or down the stairs, switch the on/off switch to the on position, and use the control to start the stairlift.

To use the seatbelt


Your stairlift may be fitted with one of the seatbelts shown below. Pull the seatbelt across you and fasten together.

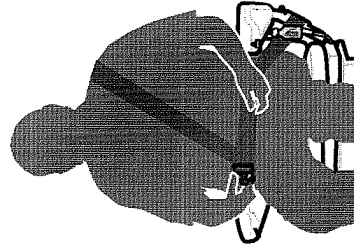

Diagonal side fastening seatbelt

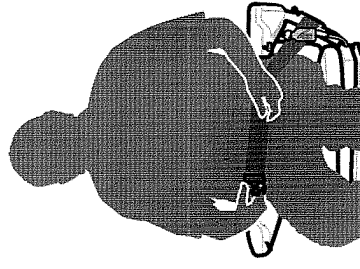

Side fastening seatbelt

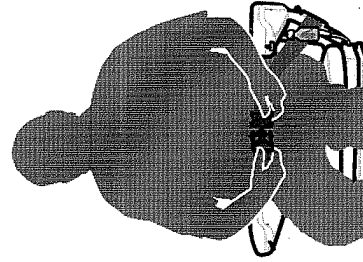

Basic centre fastening seatbelt

Safety

Always use the seatbelt.

Only undo it when you have reached the top or bottom of the stairs and are ready to get off.

Keep your elbows tucked in close to your body when using the stairlift.

Only one person should ever use the stairlift at any time. Maximum weight is 120kg (approx 19 stone).


To start the stairlift

Your stairlift may be fitted with a joystick or a push button control depending on which you chose.


To use the joystick: move and hold the joystick in the direction you want to travel.

To use the push button: press and hold the button in the direction you want to travel


The stairlift will start after a few seconds – the delayed start is a safety feature. When the stairlift has reached the top or bottom of the stairs, it will stop. If you release the control at any point, the stairlift will stop.

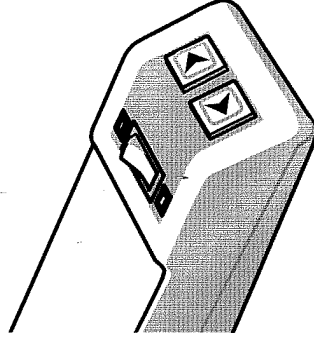


Sarum push button

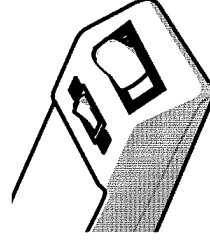

Sarum joystick

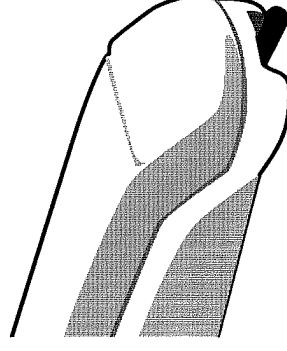

Saxon joystick

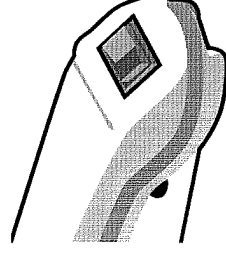

Saxon push button

Safety

Never use your stairlift in the event of fire.

Using your stairlift




4 Swivel the seat

At the top of the stairs you can swivel the seat towards the landing to help you get off the stairlift. You can also swivel the seat to make it easier to get on the stairlift when you want to go back down.

To swivel the chair using the standard swivel seat

Push the swivel levers down fully on the sides of the seat and swivel the seat to the side.


You can only do this in the direction of the top of the stairs – this is a safety feature.

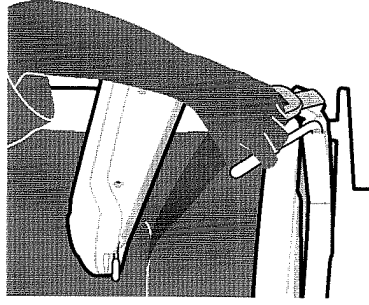

To swivel the chair using the automatic swivel seat (optional)

Push and hold the joystick in the direction you want to swivel the seat.

You can also use the buttons on the remote control to swivel the seat – press and hold the up arrow.

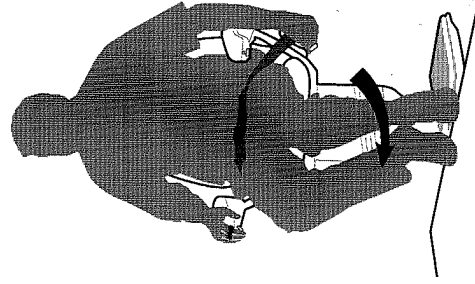


Using the hinged rail

Your stairlift may be fitted with a hinged rail that you can raise so the rail does not obstruct any doorways or passageways. The stairlift will not work until the hinged rail is fully lowered.

To use the automatic hinged rail (optional)

Make sure the chair is at the top of the stairs.


Press and hold the arrow button on the hinge control until the hinge is fully raised or lowered.

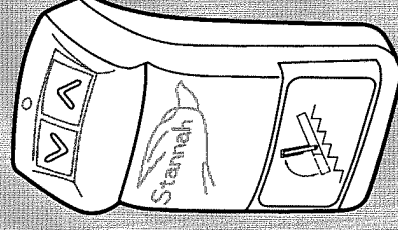

To use the standard hinged rail (optional)

Make sure the chair is at the top of the stairs.

Use the underside of the rail to raise or lower the hinged section of the rail.

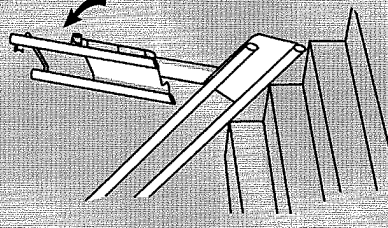

Safety

Take care when raising or lowering the hinged rail.